

putting ON PAPR

Donning Checklist 1/2

PAPR system




coveralls



two pair long
cuff gloves



battery
pack

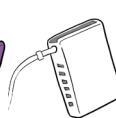

belt



gown



shoe
liners



booties



White-hooded PAPR Series 2080

Shoe liners + gloves

- 1 Qualified monitor is present.
- 2 **Set up** cart for donning and doffing.
- 3 **Remove** personal effects (phone, keys, badges, pens, etc.).
- 4 **Put on** clear inner shoe liners over your shoes and pants.
- 5 **Put on** extended cuff inner gloves.

Belt + helmet

- 6 **Secure** belt snug around waist with clip facing up. **Place** excess belt into clip and loop.
- 7 **Secure** the battery on the left side of the belt.
- 8 **Connect** helmet to battery.
- 9 **Check** that 3 LED lights on helmet are green to ensure battery is fully charged.
- 10 **Adjust** temple holder (top hole for larger head, 2nd hole works for most though).
- 11 **Put** helmet on head and **size** to fit by turning ratchet knob to adjust the headband.
- 12 **Bend** over to ensure helmet is tight enough to stay on without slipping.
- 13 **Ensure** long hair is in a low bun or ponytail is above shoulders.
- 14 **Adjust** fan speed to comfort.
- 15 **Disconnect** the battery.

Hood + cap

- 16 **Open** the hood bag.
- 17 **Set aside** outer cap.
- 18 **Orient** the hood so markings face you and are in upward direction and flat edge is at top.
- 19 **Hold** the upper corners and **shake** the hood until it fully unfolds.
- 20 **Connect** the center hole of the hood to the center hook on the front of the helmet. Once you are hooked, **keep pushing** until you hear a snap.
- 21 **Connect** the right and left sides of the hood to the helmet in the same way until you hear a snap.
- 22 **Grab** the bottom of the hood and **pull** it back and over the helmet until it fits.
- 23 **Take** the cap and **fit it** fully over the top of the hood.
- 24 **Set** assembled PAPR hood aside.

Coveralls + booties

- 25 **Put on** coveralls outside room. Feet in first. Open-end faces your front. Do not zip up yet.
- 26 **Ensure** proper mobility of arms and legs within coveralls.
- 27 **Tape** coverall and wrist of the gloves and **form** a release tab at end of the loop.
- 28 **Don** booties.

continued next page...

PAPR Donning Checklist 2/2

Helmet + zip + hood

- 29 **Sanitize** gloves and touched dispenser.
- 30 **Insert** PAPR power cord connector into the battery.
- 31 **Put** the helmet on, letting the hood drape around your shoulders over the coveralls. Last chance to **adjust** the helmet!
- 32 **Tuck** the cord into your suit.
- 33 **Zip up** the front of the coveralls.
- 34 **Insert** fingers into facial liner and **tug** it around your face until it tucks under your chin.
- 35 **Find** both neck ties and bring them to the front and **tie** a shoelace knot close to neck.
- 36 **Find** both torso ties from center back of hood, bring them around and **insert** them through the front holes, then **tie** into shoelace knot.

Gown + outer gloves

- 37 **Unfold** and **put on** gown.
- 38 **Secure** neck of gown, making sure neck tie of hood is covered.
- 39 **Tie** gown at waist at left side.
- 40 **Put on** extended cuff outer gloves, **tuck** gown sleeves into gloves.
- 41 Safety monitor completes **360° check** of PPE.

Entry

- 42 **Sanitize** Gloves and touched dispenser.
- 43 **Enter** room. Do not enter room without proper fit of the PAPR system.
- 44 **Do not touch** or re-adjust PAPR system inside the room.

PAPR Doffing Checklist 1/2

Inside room

Setup

- 1 **Make sure** that you have everything you need to complete a successful doffing. (Trash can, hand sanitizer, balance aid).
- 2 **Check** that your partner outside of the room is aware you are preparing to doff.
- 3 **Inspect** PPE for visible signs of contamination. Have your **partner inspect** your back through the window on the door.
- 4 **Wipe away** contamination with SaniWipes.
- 5 **Place** a large underpad in front of the door, with trash can and sanitizer within reach.
- 6 **Sanitize** gloves and if touched, dispenser.

Cap + gown

- 7 With one hand, reach up to the back of the helmet, **pinch** and **remove** cap from top of PAPR hood, throw into trash.
- 8 **Sanitize** gloves and if touched, dispenser.
- 9 Gently **pull** the sleeves of the gown out from under each outer glove cuff, working your way around the wrist.
- 10 **Sanitize** gloves and if touched, dispenser.
- 11 **Grasp** gown at sides over waist tie, **pull forward** to break tie.
- 12 **Cross** your arms in an "X" to **grasp** gown at upper arm, close to shoulder, **avoid** touching PAPR hood.
- 13 **Uncross** your arms slowly, extending them away from your body, inverting the gown. **Extend** your arms outward until they are straight.
- 14 Keeping arms extended away from your body, **bundle** the gown and discard.
- 15 **Sanitize** gloves and if touched, dispenser.

Booties

- 16 Using balance aid if appropriate, with one hand **remove** one bootie. As you remove it, step with that foot onto pad. Throw bootie away.
- 17 **Remove** other bootie. Step onto pad. Throw bootie away.

Outer gloves + hood + tape

- 18 **Sanitize** gloves and if touched, dispenser.
- 19 **Pinch** glove cuff and pull over fingers, but do not remove glove. With covered fingers, **pinch** and **pull** opposite glove off. Using other hand, **remove** initial glove and discard both gloves.
- 20 **Sanitize** gloves and touched dispenser.
- 21 **Untie** torso ties, **ball up** and **pull through** one side, and discard.
- 22 **Sanitize** gloves and touched dispenser.
- 23 **Untie** neck ties, **ball up** and **pull through** one side, and discard.
- 24 **Sanitize** gloves and if touched, dispenser.
- 25 In a controlled manner, **pull** tape down and away to remove tape from both wrists.

Coveralls

- 26 **Sanitize** gloves and if touched, dispenser.
- 27 Avoiding touching the inside of the PAPR hood, **zip down** coveralls with one hand.
- 28 **Grasp** the coveralls near lower back, at the hips, and **pull** the coveralls straight down.
- 29 Touching only the outside of the coveralls, **push** the coveralls down below the knees.
- 30 **Step on** the crotch of the suit to remove feet from coverall. **Repeat** step motion until feet are free.

Exit

- 31 **Step off** coveralls and leave the coveralls for the next person to pick up.
- 32 **Sanitize** gloves and touched dispenser.
- 33 Observer **opens** the door.
- 34 **Exit** the patient room.

Continue: "OUTSIDE room"

PAPR Doffing Checklist 2/2

Outside room

Hood + inner gloves

35 **Sanitize** gloves and if touched, dispenser.

36 **Place** both hands under the front of the hood, palms forward. **Do not cross arms. Reach** to the back of the helmet on each side. Fully **gather** hood material and **tightly roll** touching only the inside of the hood.

37 Continue to **roll** the hood material up across the back and top of the helmet until you reach the top of the head, in line with the ears.

38 **Move** fingers to the edge of the face shield and unsnap the temple post from the helmet and fold visor in half.

39 With an upward motion, **pull** folded visor from center post.

40 **Remove** hood.

41 **Sanitize** gloves and if touched, dispenser.

42 **Pinch** glove cuff and **pull** over fingers, but **do not remove** glove. With covered fingers, **pinch** and **pull** opposite glove off. Using other hand, **remove** initial glove and **discard** both gloves.

shoe covers + belt

43 **Sanitize** hands and if touched, dispenser.

44 **Raise** arms over head in "raise the roof" motion.

45 Observer **holds** battery.

46 Observer **unbuckles** belt.

47 Observer **removes** helmet motor from behind, **avoiding** the face.

Clean up

48 **Put on** clean gloves

49 **Remove** shoes covers

50 **Pinch** glove cuff and **pull** over fingers, but **do not remove** glove. With covered fingers, **pinch** and **pull** opposite glove off. Using other hand, **remove** initial glove and **discard** both gloves.

51 **Sanitize** hands and if touched, dispenser.

52 **Wash** hands and arms thoroughly with soap and water.

53 **Talk** with your infection prevention team about showering protocols.