

putting PPE ON 1/1



Coveralls



CAPR system

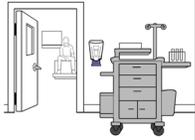


Inner gloves

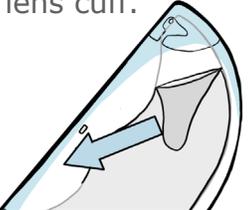
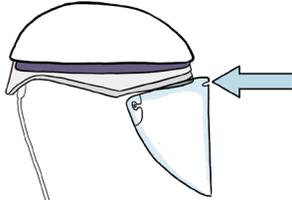
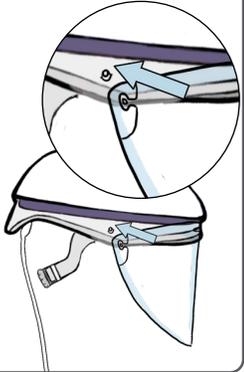
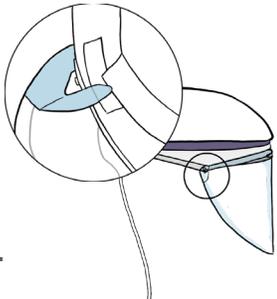
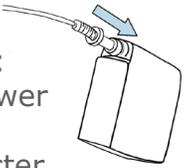
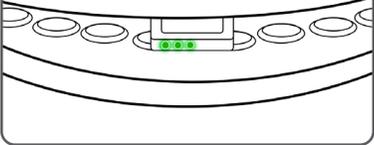


Outer gloves

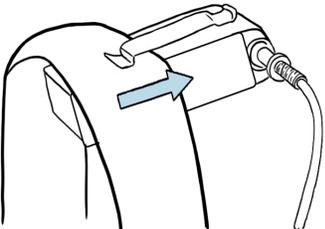
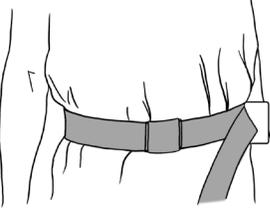
Gloves

- 1** **Batch work and plan ahead**, having items available outside each patient room. 
- 2** If you already have a mask on:  **Sanitize hands.**  **Remove mask.**  **Store mask.**  **Sanitize hands.**
- 3**  **Put on first pair of gloves.**

Lens Cuff

- 4** **Remove** the lens protector liner from the front side of the lens cuff. 
- 5** **Secure** the lens cuff by **aligning** the center hole over the center attachment post on the helmet. 
- 6** **Attach** the left and right side lens cuff mounting holes over the left and right side attachment posts of the helmet. 
Tip: The lens cuff will click when it is properly secured.
- 7** **Lift** the cuff flappers to rest against the foam pads at the ends of the comfort strip. 
- 8** **Insert** the power cord connector into the battery. 
- 9** **Check** the LED lights to ensure the battery is fully charged. 
- 10** **Adjust** fan speed to comfort. 

Battery, Belt

- 11** **Secure** the battery to the belt using the battery clip. 
- 12** **Secure** the belt around the waist. 



CAPR, coveralls, double gloves

Helmet

13 **Turn** ratchet knob counterclockwise to loosen the headband.

14 **Hold** the helmet near the rear headband with one hand, pull down on top of the lens cuff with the other.

15 **Place** the chin into the lens cuff while pulling the helmet over and down on the head.

16 **Turn** the ratchet knob clockwise to tighten the headband.

17 **Slide** the fingers around the cuff under the chin to **ensure** slight tension.

18 **Check** for proper fit, ensure:

- The cuff flappers are perpendicular to the head.
- The bottom of the headband rests 1/2 inch from eyebrows.
- LED indicators are in your peripheral vision.
- The cuff flappers are 1/4 inch from the temples.

19 **Tip:** If the headband is not in an appropriate height, doff, adjust the headband height position, and redo.

Entry

20 **Put on** coveralls outside room. Open-end **faces** your front. **Zip up** the front of the coveralls.

21 **Sanitize** first pair of gloves.

22 **Put on** second pair of gloves.

23 **Sanitize** second pair of gloves.

24 **ENTER room**
DO NOT enter the room if you do not achieve a proper fit of the CAPR system.

25 **Do not touch or re-adjust** CAPR system inside the room.



CAPR, coveralls, double gloves

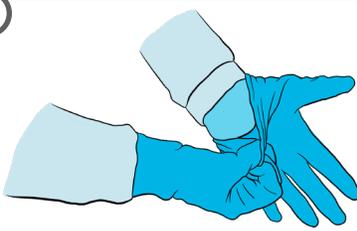
Next patient: COVID+

Gloves

1 **Sanitize** outer gloves.



2 **Remove** outer gloves.

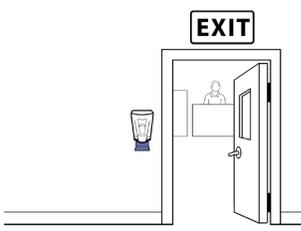


3 **Sanitize** inner gloves.



Exit

4 **Exit** patient room.





CAPR, coveralls, double gloves

Next patient: COVID+

Gloves

After outer glove removal in the patient room, exit and...

1 **Sanitize** inner gloves.

A pair of hands wearing blue gloves is being held under a white hand sanitizer dispenser.

2 **Put on** fresh outer gloves.

A hand is shown pulling a new blue glove onto the wrist.

Walking

3

A person in full PPE (CAPR, coveralls, gloves, and face shield) is walking in a hallway. A dashed line on the floor indicates a marked zone. An arrow points forward.

Walk to next patient with confirmed COVID-19 (marked) while **remaining** in marked zone.

4

A person in PPE is walking in a hallway. Three circular icons with red X's over them show prohibited actions: touching the face, touching the walls/rails, and touching other people.

Do not touch your face, the walls, rails, or anybody else until reaching the next room.

Entry

5 **Sanitize** outer gloves.

A pair of hands wearing blue gloves is being held under a white hand sanitizer dispenser.

6

Enter NEXT COVID-19 patient room or area.

A person in PPE is entering a room. A sign above the door says "ENTER". Inside the room, another person is sitting at a desk. A hand sanitizer dispenser is on the wall.

7 **Do not** touch face or re-adjust CAPR system inside room.

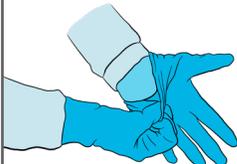
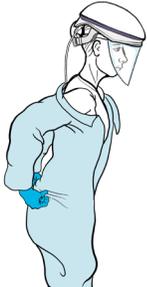
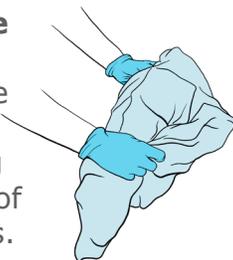
A person in PPE is shown with red X's over their face and hands, indicating that touching the face or adjusting the CAPR system is prohibited.



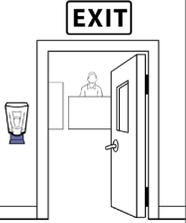
CAPR, coveralls, double gloves

Next patient: NOT COVID+

Gloves, Coveralls

- 1 Sanitize** outer gloves. 
- 2 Remove** outer gloves. 
- 3 Sanitize** inner gloves. 
- 4 Wipe down** torso, arms, and legs. 
- 5 Sanitize** inner gloves. 
- 6 Zip down** coveralls. 
- 7 Grasp** coveralls around lower back area and **pull** down. 
- 8 Remove** coveralls from shoulders and **pull** down below knees. 
- 9 Remove and step out** of the coveralls by marching and stomping on the arms of the coveralls. 
- 10 Remove** into bin for reuse without touching outside of coveralls. 
- 11 Sanitize** gloves. 

Exit

- 12 Exit** patient room. 

The materials are intended solely for general educational and information purposes, are made available in the context of the public health emergency related to the coronavirus (COVID-19) and have not been subject to review that typically would occur in a non-emergent situation. The materials do not constitute the provision of medical, legal or other professional advice.



CAPR, coveralls, double gloves

Next patient: NOT COVID+

Lens Cuff

After coveralls and outer glove removal...

1 **Sanitize** gloves.

2 **Place** first wipe on table.

3 **Grab** lens cuff at the left and right side attachments. **Remove** and by pulling outward and forward.

4 **Place** lens cuff on wipe.

5 **Wipe** front and back of lens cuff. **Allow** to dry.

6 **Store** lens cuff in a gallon sized bag labeled with your name.

Helmet

7 **Sanitize** gloves.

8 **Place** second wipe on table.

9 Grasp outer-shell of helmet and **take off**. **Ensure** that the helmet cord doesn't touch your body.

10 **Place** helmet on first wipe.

11 **Sanitize** gloves.

12 **Put on** mask.

13 **Wipe** inside of the helmet.

14 **Sanitize** gloves.

15 Hold inside of the helmet and **flip** onto second wipe.

16 **Wipe** outside of the helmet.

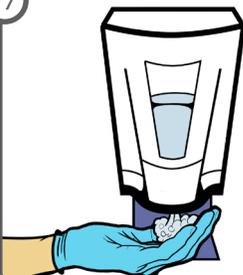


CAPR, coveralls, double gloves

Next patient: **NOT COVID+**

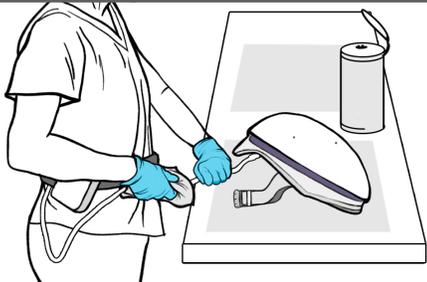
Battery

17



Sanitize gloves.

18



Wipe helmet cord.

19



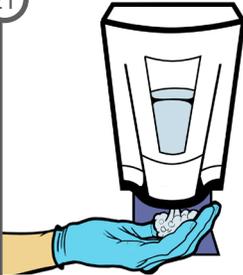
Sanitize gloves.

20



Remove and wipe battery.

21



Sanitize gloves.

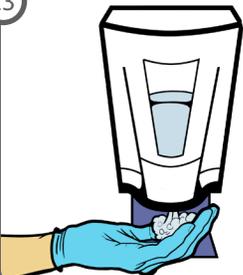
22



Remove belt and wipe.

Gloves

23



Sanitize gloves.

24



Remove gloves.

25



Wash hands.

The materials are intended solely for general educational and information purposes, are made available in the context of the public health emergency related to the coronavirus (COVID-19) and have not been subject to review that typically would occur in a non-emergent situation. The materials do not constitute the provision of medical, legal or other professional advice.