

# putting PPE ON <sup>1/2</sup>



Launderable gown



CAPR system



Inner gloves



Outer gloves

## Gloves

1

**Batch work and plan ahead**, having items available outside each patient room.



2

If you already have a mask on:



**Sanitize hands.**



**Remove mask.**



**Store mask.**



**Sanitize hands.**

3

**Put on first pair of gloves.**



## Lens Cuff


4

**Remove** the lens protector liner from the front side of the lens cuff.




5

**Secure** the lens cuff by **aligning** the center hole over the center attachment post on the helmet.

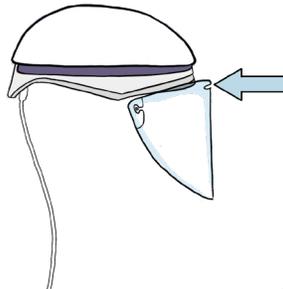

6


**Attach** the left and right side lens cuff mounting holes over the left and right side attachment posts of the helmet.

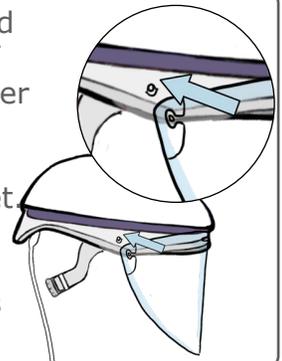

*Tip:* The lens cuff will click when it is properly secured.


7

**Lift** the cuff flappers to rest against the foam pads at the ends of the comfort strip.

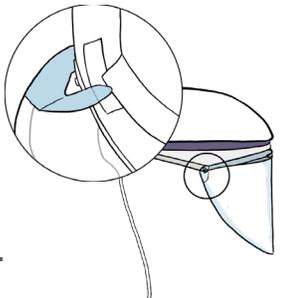


8

**Insert** the power cord connecter into the battery.

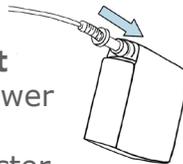

9

**Check** the LED lights to ensure the battery is fully charged.

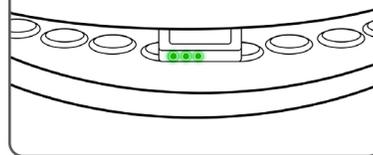

10


**Adjust** fan speed to comfort.



## Battery, Belt


11

**Secure** the battery to the belt using the battery clip.

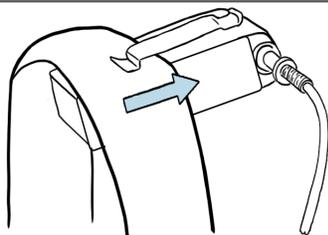

12

**Secure** the belt around the waist.

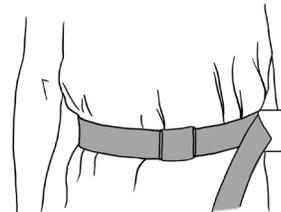



**CAPR, launderable gown, double gloves**

# Helmet

13 **Turn** ratchet knob counterclockwise to loosen the headband.

14 **Hold** the helmet near the rear headband with one hand, pull down on top of the lens cuff with the other.

15 **Place** the chin into the lens cuff while pulling the helmet over and down on the head.

16 **Turn** the ratchet knob counterclockwise to tighten the headband.

17 **Slide** the fingers around the cuff under the chin to **ensure** slight tension.

18 **Check** for proper fit, ensure:

- The cuff flappers are perpendicular to the head.
- The bottom of the headband rests 1/2 inch from the eyebrows.
- LED indicators are in your peripheral vision.
- The cuff flappers are 1/4 inch from the temples.

19 **Tip:** If the headband is not in an appropriate height, doff, adjust the headband height position, and redo.

# Gown, Gloves

20 **Put on** gown outside room. Open-end **faces** your back. **Tie** the back of the gown.

21 **Pull** cuffs half way over palm of the inner glove.

22 **Put on** second pair of gloves.

23 **Sanitize** second pair of gloves.

24 **ENTER room** **DO NOT** enter the room if you do not achieve a proper fit of the CAPR system.

25 **Do not** touch or re-adjust **CAPR** system inside the room.



**CAPR, launderable gown, double gloves**

**Next patient: COVID+**

## Gloves

1 **Sanitize** outer gloves.

2 **Remove** outer gloves.

## Exit

3 **Exit** patient room.



CAPR, launderable gown, double gloves

**Next patient: COVID+**

**Gloves**

After outer glove removal in the patient room, exit and...

1 **Pull** cuffs half way over palm of the inner glove.




2 **Put on** fresh outer gloves.




**Walking**

3 **Walk** to next patient with confirmed COVID-19 (marked) while **remaining** in marked zone.

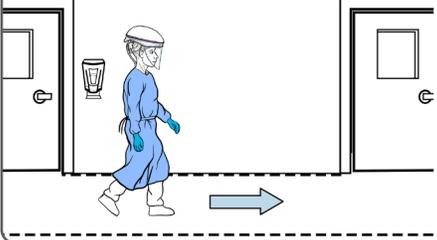

4 **Do not touch** your face, the walls, rails, or anybody else until reaching the next room.

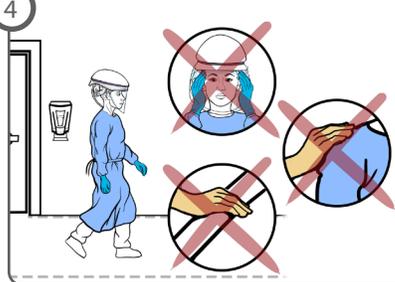


**Entry**

5 **Sanitize** outer gloves.



6 **Enter** next COVID-19 patient room or area.

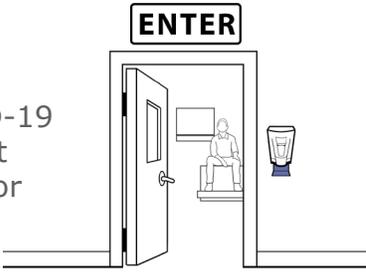

7 **Do not touch** face or re-adjust CAPR system inside room.





CAPR, launderable gown, double gloves


Next patient: **NOT COVID+**

## Gown, Gloves

1 **Sanitize** outer gloves.



2 **Remove** outer gloves.

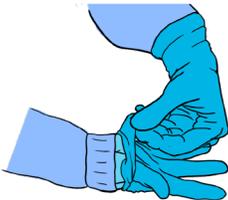

3 **Sanitize** inner gloves.



4 **Reach back and release** the neck portion.



5 **Untie** the waist.



6 **Remove** the gown into the designated bin.




7 **Sanitize** inner gloves.



## Exit

8 **Exit** patient room.



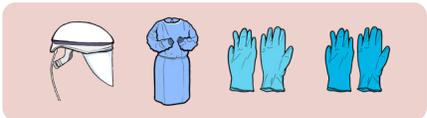

**CAPR, launderable gown, double gloves**

**Next patient: NOT COVID+**

## Lens Cuff

After coveralls and outer glove removal...

1 **Sanitize** gloves.

2 **Place** first wipe on table.

3 **Grab** lens cuff at the left and right side attachments. **Remove** by pulling outward and forward.

4 **Place** lens cuff on wipe.

5 **Wipe** front and back of lens cuff. **Allow** to dry.

6 **Store** lens cuff in a gallon sized bag labeled with your name.

## Helmet

7 **Sanitize** gloves.

8 **Place** second wipe on table.

9 Grasp outer-shell of helmet and **take off**. **Ensure** that the helmet cord doesn't touch your body.

10 **Place** helmet on first wipe.

11 **Sanitize** gloves.

12 **Put on** mask.

13 **Wipe** inside of the helmet.

14 **Sanitize** gloves.

15 Hold inside of the helmet and **flip** onto second wipe.

16 **Wipe** outside of the helmet.



CAPR, launderable gown, double gloves

Next patient: **NOT COVID+**

### Battery, Belt

17

Sanitize gloves.

18

Wipe helmet cord.

19

Sanitize gloves.

20

Remove and wipe battery.

21

Sanitize gloves.

22

Remove belt and wipe.

### Gloves

23

Sanitize gloves.

24

Remove gloves.

25

Wash hands.

The materials are intended solely for general educational and information purposes, are made available in the context of the public health emergency related to the coronavirus (COVID-19) and have not been subject to review that typically would occur in a non-emergent situation. The materials do not constitute the provision of medical, legal or other professional advice.