










Inner gloves Outer gloves

#### ,

### **Gloves**

**Adjust Helmet** 

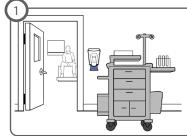

**Batch work** and **plan ahead**, having items available outside each patient room.






Secure the belt around the waist.




Secure the battery on the left side of the belt.

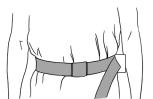

Connect the helmet to the battery.


**Check** the LED lights to ensure the battery is fully charged.

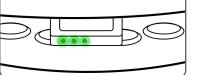


Put helmet on head and size to fit by turning ratchet knob to adjust the headband.



Adjust fan speed to comfort.

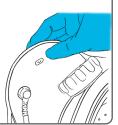

Disconnect the battery.

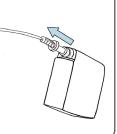


**Take off** the helmet and **place** it inside the PAPR hood.

**Ensure** the white HEPA filter is on top, the helmet is inside filter pouch, and the adjustable knob is outside of the filter pouch.



# **Put on Hood**

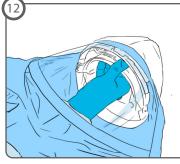

**Slide** the helmet into the hood until it snaps in the center hole. **Snap** helmet into the hood on the left and right side snaps of the face shield.



**Set** the assembled PAPR hood **aside**.









# Gown, Gloves, PAPR



Put on gown outside room.

Open-end faces your back.

**Tie** gown at the neck.





**Insert** PAPR power cord connector into the battery.



**Tie** gown at waist.

Put the helmet on, letting the hood drape around your shoulders over the gown.



**Straighten** the hood draping and **tie** it down.





Put on second pair of gloves.



## **Entry**

ENTER room

**DO NOT enter** the room if you do not achieve a proper fit of the PAPR system.



**Do not** touch or re-adjust **PAPR** system inside the room.













Next patient: COVID+


## **Patient Room**







## **Exit**

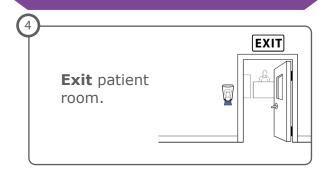









# OUTSIDE

Next patient: COVID+


## Gloves

After outer glove removal in the patient room, exit and...






# Walking

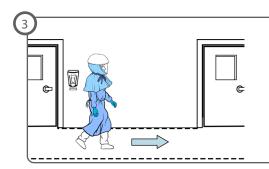

Walk to next patient with confirmed COVID-19 (marked) while remaining in marked zone.

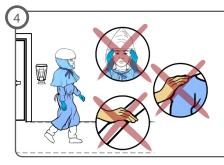


Do not touch your face, the walls, rails, or anybody else until reaching the next room.

### **Entry**





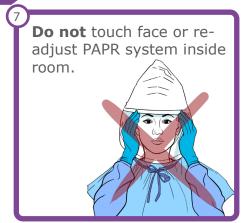









# INSIDE

Next patient: NOT COVID+

#### Gown









With your back to the exit, have a buddy stand outside the room and do the next two steps while wearing clean gloves.









#### **Exit**











# OUTSIDE

Next patient: **NOT** COVID+

#### Hood

After gown and outer glove removal...









Pull face shield forward and release helmet pin.



Grab hood top, pull back, then pull forward to remove hood, avoiding your face.



If reusing, place hood in designated bin.

# **Helmet, Belt, Battery**









Remove and wipe belt.











