

# putting PPE ON 1/1



Coveralls



PAPR system



Inner gloves



Outer gloves

## Gloves

- Batch work and plan ahead**, having items available outside each patient room.
- If you already have a mask on:

**Sanitize** hands. **Remove** mask. **Store** mask. **Sanitize** hands.
- Put on** first pair of gloves.

## Helmet

- Secure** the belt around the waist.
- Secure** the battery on the left side of the belt.
- Connect** the helmet to the battery.
- Check** the LED lights to ensure the battery is fully charged.
- Put** helmet on head and **size** to fit by turning ratchet knob to adjust the headband.
- Adjust** fan speed to comfort.
- Disconnect** the battery.
- Take off** the helmet and **place** it inside the PAPR hood.

**Ensure** the white HEPA filter is on top, the helmet is inside filter pouch, and the adjustable knob is outside of filter pouch.

## Hood

- Slide** the helmet into the hood until it snaps in the center hole. **Snap** helmet into the hood on the left and right side snaps of the face shield.
- Set** the assembled PAPR hood **aside**.



**PAPR, coveralls, double gloves**

**Coveralls, Gloves, PAPR**

14 **Sanitize** first pair of gloves.

15 **Put on** coveralls outside room.  
Open-end **faces** your front.

16 **Insert** PAPR power cord connector into the battery.

17 **Put** the helmet on, letting the hood drape around your shoulders over the coveralls.

18 **Zip up** the front of the coveralls.

19 **Straighten** the hood draping and **tie** it down.

20 **Put on** second pair of gloves.

21 **Sanitize** second pair of gloves.

**Entry**

22 **ENTER room** **DO NOT** enter the room if you do not achieve a proper fit of the PAPR system.

23 **Do not** touch or re-adjust **PAPR** system inside the room.



**PAPR, coveralls, double gloves**


**Next patient: COVID+**

## Gloves

1 **Sanitize** outer gloves.



2 **Remove** outer gloves.

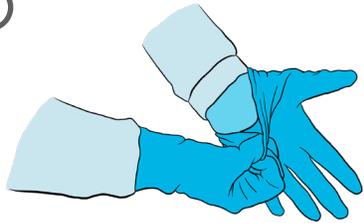


**Remove** outer gloves.

3 **Sanitize** inner gloves.



## Exit

4 **Exit** patient room.

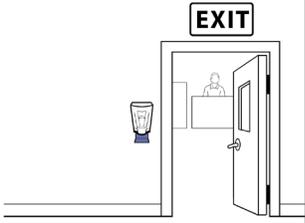



**PAPR, coveralls, double gloves**

**Next patient: COVID+**

**Gloves**

After outer glove removal in the patient room, exit and...

1 **Sanitize** inner gloves.

An illustration showing a pair of hands being held under a white hand sanitizer dispenser.

2 **Put on** fresh outer gloves.

An illustration showing a person's hands being put on a new pair of blue gloves.

**Walking**

3 **Walk** to next patient with confirmed COVID-19 (marked) while **remaining** in marked zone.

An illustration of a person in full PPE walking in a hallway. A dashed line on the floor indicates a marked zone, and an arrow points forward.

4 **Do not touch** your face, the walls, rails, or anybody else until reaching the next room.

An illustration of a person in PPE walking in a hallway. Three circular icons with red X's over them show prohibited actions: touching the face, touching the wall, and touching another person.

**Entry**

5 **Sanitize** outer gloves.

An illustration showing a pair of hands being held under a white hand sanitizer dispenser.

6 **Enter** NEXT COVID-19 patient room or area.

An illustration of a person in PPE entering a room. A sign above the doorway says "ENTER".

7 **Do not touch** face or re-adjust PAPR system inside room.



An illustration of a person in PPE. Two circular icons with red X's over them show prohibited actions: touching the face and adjusting the PAPR hood.



**PAPR, coveralls, double gloves**

**Next patient: NOT COVID+**

## Gloves, Coveralls

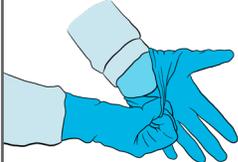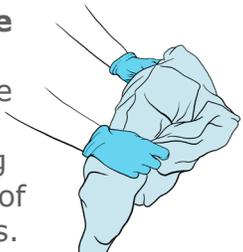- 1 Sanitize** outer gloves. 
- 2 Remove** outer gloves. 
- 3 Sanitize** inner gloves. 
- 4 Wipe down** torso, arms, and legs. 
- 5 Sanitize** inner gloves. 
- 6 Zip down** coveralls. 
- 7 Grasp** coveralls around lower back area and **pull down**. 
- 8 Remove** coveralls from shoulders and **pull down** below knees. 
- 9 Remove and step out** of the coveralls by marching and stomping on the arms of the coveralls. 
- 10 Remove** into bin for reuse without touching outside of coveralls. 
- 11 Sanitize** gloves. 

## Exit

- 12 Exit** patient room. 

The materials are intended solely for general educational and information purposes, are made available in the context of the public health emergency related to the coronavirus (COVID-19) and have not been subject to review that typically would occur in a non-emergent situation. The materials do not constitute the provision of medical, legal or other professional advice.



**PAPR, coveralls, double gloves**

**Next patient: NOT COVID+**

**Hood**

After gown and outer glove removal...

- Sanitize** gloves.
- Place** wipe on table.
- Untie** the PAPR hood.
- Grasp** face shield at the sides and **pull out** snaps.
- Pull** face shield forward and **release** helmet pin.
- Grab** hood top, **pull back**, then **pull forward** to **remove** hood, avoiding your face.
- If reusing, **place** hood in designated bin.

**Helmet, Belt, Battery**

- Sanitize** gloves.
- Remove** helmet motor, **DO NOT** touch your face.
- Put on** mask.
- Wipe** helmet.
- Remove** and **wipe** battery.
- Remove** and **wipe** belt.
- Sanitize** gloves.
- Remove** gloves.
- Wash** hands.

The materials are intended solely for general educational and information purposes, are made available in the context of the public health emergency related to the coronavirus (COVID-19) and have not been subject to review that typically would occur in a non-emergent situation. The materials do not constitute the provision of medical, legal or other professional advice.