

# store your N95

## What you need

View the PPE videos at [med.emory.edu/PPE](https://med.emory.edu/PPE)



N95 Respirator



Paper storage bag with handles

## Removal

- 

**Sanitize** gloves.
- 

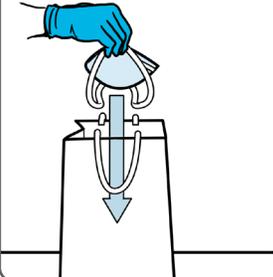
**Pinch** bottom strap and **pull** far over head.  
**Do not** let straps touch your face.
- 

**Pinch** top strap and **pull** far over head.  
**Do not** let straps touch your face as you **remove** the N95.
- 

**Position** and hold N95 facedown.

## Storage

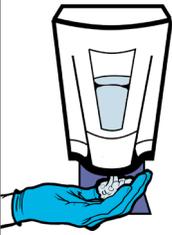
- 

**Place** storage bag on clean, flat surface.
- 

**Place** N95 facedown into the storage bag.  
**Avoid** touching inside or outside of bag.
- 

**Sanitize** gloves.

## Retrieval

- 

**Sanitize** gloves.
- 

**Remove** N95 from the storage bag.  
**Avoid** touching inside or outside of bag.
- 

**Put on** N95, ensuring proper seal.  
**Ensure** straps are not crossing.
- 

**Sanitize** gloves.