


Safely donning and doffing Cloth Gowns



Reusable gown, Gloves

Donning

1

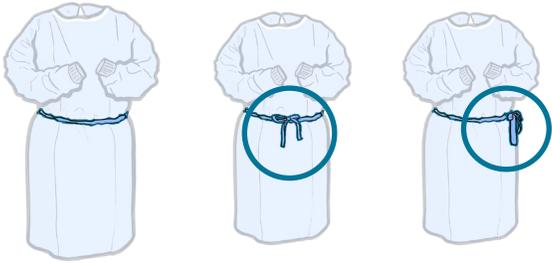

Where the isolation gown is secured may vary by manufacturer.

Note: Reusable gowns are designed to be untied and not torn off.

2 Do not double knot.



3 Pull down sleeves to around the palm.



4 Put on gloves. Ensure the glove covers the sleeves.



Doffing

5 Sanitize gloves.



6 With your back to the exit, have buddy do the next two steps while wearing clean gloves. The buddy should avoid touching the gown with their scrubs.



7 Release the neck portion.




8 Untie waist.



9 Roll the gown towards your hands. Remove the gown with the gloves into the designated bin.



10 Sanitize hands.

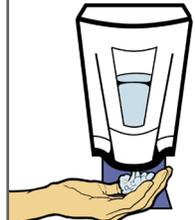

For full donning and doffing instructions, please visit med.emory.edu/PPE