


Trimming Your Face Shield

Templates and videos courtesy the EUH IP Team.


- 1 Open videos and review.




- 2 Print out templates if desired – tape together to make a full size template.

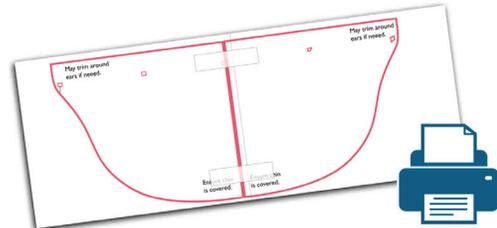


- 3 Align holes of plastic shield with the marks on the template.

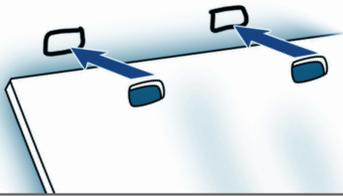

- 4 Trace line on shield with sharpie or magic marker.

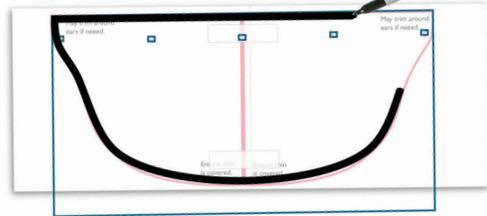

- 5 Cut shield by following the traced line.

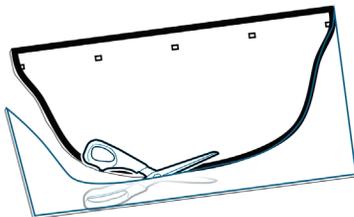

- 6 Apply to headband to check fit.



The shield should be trimmed to just below chin and to just in front of ears.



Ensure chin
is covered.



bring this edge together



May trim around
ears if needed.



bring this edge together

Ensure chin
is covered.

May trim around
ears if needed.

